

## How to Update the QUICKCHECK<sup>webline</sup> Firmware

### NOTE

This technical note describes how to update the firmware of your QUICKCHECK<sup>webline</sup>. PTW recommends downloading the saved measurement data from your QUICKCHECK<sup>webline</sup> to your PC before performing the update to the new firmware version.


1. Disconnect the power cable from your QUICKCHECK<sup>webline</sup>.
2. Make sure the device is turned off.
3. Connect the QUICKCHECK<sup>webline</sup> to your PC via RS232.
4. Start the **QUICKCHECKwebline Updater** on your PC.
5. Reconnect the power cable to your QUICKCHECK<sup>webline</sup>.
6. A message is displayed in the **QUICKCHECKwebline Updater** window, informing you that a connection to the QUICKCHECK<sup>webline</sup> has been established:



If the message is not displayed, try selecting a different COM port. Disconnect the power cable and turn off the QUICKCHECK<sup>webline</sup>. Make sure the RS232 connection between your PC and the device is established. Reconnect the power cable.

7. Click **OK**.
8. A new message is displayed in the **QUICKCHECKwebline Updater** window, informing you that the update procedure is irreversible. Confirm that you wish to start the updating process by clicking the **Start update** button.

9. The QUICKCHECK *webline* firmware will be updated. Do not close the **QUICKCHECKwebline Updater** window.
10. Once the update is complete, the message **Update successful** is displayed in the **QUICKCHECKwebline Updater** window.

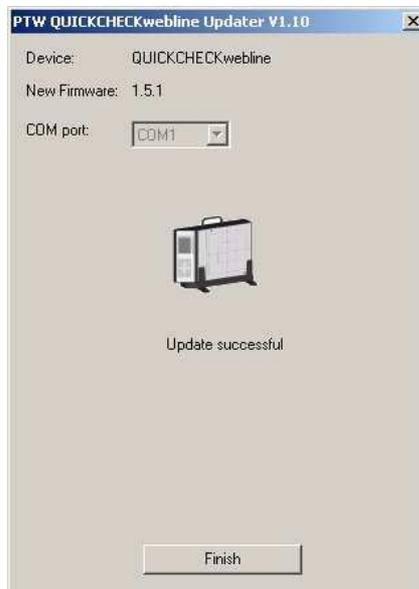

11. Click the **Finish** button to close the window. Your QUICKCHECK *webline* firmware has been successfully updated.

---

PTW-Freiburg  
Lörracher Straße 7 • 79115 Freiburg • Germany  
Phone +49 761 49055-0 • Fax +49 761 49055-70  
info@ptw.de • www.ptw.de