Regular Maintenance of the Print Head of "CITIZEN CT-S281"

CAUTION

This document is a supplement to the user manual of the "CITIZEN CT-S281" printer. Use the document only in combination with the "CITIZEN CT-S281" user manual.

Follow the General Information and Safety Information given in the user manuals.

General

While normal use of the "CITIZEN CT-S281" together with the paper roll (Art. Nr. 7401-DP050.02) continuous abrasion of the silicone layer on the paper roll can occur and lead to wrong label positioning and to wrong printing. Therefore the print head requires periodical cleaning.

Clean the print head after approximately 2,000 prints or at latest when printing errors occur.

Perform cleaning according to the description in section 4.3 of the printer user manual which is attached to the printer or which can be downloaded from <http://www.citizen-systems.com/Pages/UisSupport/user-manuals.aspx>¹.

Cleaning

CAUTION

The print head is hot immediately after printing.

- Do not touch it with your hand.
- Do not touch the heating element of the head with a bare hand or metal object either.

- Turn the printer power off.
- Open the printer cover.
- Wait several minutes.
- Wipe off any debris on the printer head as indicated by the images using a cotton swab soaked in ethyl alcohol.

PTW-Freiburg
Lörracher Straße 7 • 79115 Freiburg • Germany
Phone +49 761 49055-0 • Fax +49 761 49055-70
info@ptw.de • www.ptw.de

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